
Table of Contents

- 1 English
- 2 Français
- 3 Español
- 4 繁体中文

BEYOND®'s custom designed Flexiblue® cheek retractor is designed to work with any whitening accelerator. The comfortable design holds lips away from enamel for the duration of the teeth whitening procedure. The innovative design includes a built in tongue depressor. Available in two sizes: medium or large.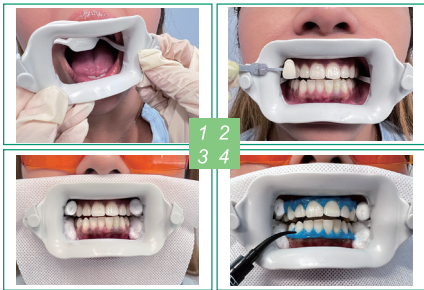

The BEYOND® FlexiPro™ Cheek Retractor is an in-office dental tool with channeled edges that comfortably fit around a patient's lips, featuring a tongue guard and bite cushion. It enlarges the mouth's work area for teeth whitening on the front teeth, protects the lips, and ensures patient comfort, with the added benefit of being cold sterilizable. Available in two sizes: medium or large.

EN

Treatment Procedure

PRE-TREATMENT

Step 1: Perform a pre-treatment oral exam and consult with the patient on his or her desired whitening results to determine the best treatment plan for his or her individual situation.

WHITENING TREATMENT

Step 2: Apply lip protection cream before inserting a cheek retractor (BEYOND® Cheek Retractor is recommended). Attach a face protection cloth, then determine the existing tooth shade. (see Fig. 1 & 2)

Step 3: Insert cotton rolls on both sides of the frenum of the upper and lower lips. Apply extra lip protection cream if necessary. (see Fig. 3)

Step 4: Dry teeth and gums. Attach preferred tip to BlueSeal syringe body (metal tip for more exacting application, plastic tip with larger aperture for quicker application). Extrude on a disposable cloth to prime before application. Apply a layer of BEYOND® BlueSeal® Gingival Protection 3-5mm wide on the gingiva. Ensure interproximal spaces are filled. Apply resin of approximately 0.5mm onto dry enamel to seal. Extend resin one tooth beyond the last tooth whitened for effective coverage. Light cure in a smooth, circular motion for around 30 seconds. (based on curing light with 800-1100mW/cm²) (see Fig. 4 & 5)

Step 5: Attach mixing tip to dual barrel syringe. (Align the large and small holes in the tip cap, insert and twist in a clockwise motion.)

Step 6: Apply a 2-3 mm layer of BEYOND® Simplex™ Chemically Activated Formula Whitening Gel to dry the surface of the front upper and lower sixteen teeth. Allow the gel to remain on the teeth for 15 minutes. (see Fig. 6)

*Note: Treatment time can be adjusted according to the patient's teeth condition and the dentist's advice.

Step 7: At the end of the 15-minute cycle, suction off the whitening gel. DO NOT rinse with water. Repeat Step 6 to apply the gel and allow the gel to remain on the teeth for 15 minutes in one more session.

Step 8: If desired results have not been achieved, apply gel and allow the gel to remain on the teeth for 10-15 minutes for the last treatment cycle (optional).

POST-TREATMENT

Step 9: At the end of the last cycle, remove the gel, gingival protection, cheek retractor, and cotton rolls. Instruct the patient to rinse his or her mouth with warm water.

Step 10: Show the patient his or her new white tooth shade. (see Fig. 7)

CONTRAINDICATIONS

1. BEYOND® Simplex™ Chemically Activated Formula Whitening Gel is contraindicated for patients with known allergies or chemical sensitivity to peroxides or glycols.
2. Do not use BEYOND® BlueSeal® Gingival Protection on patients with known sensitivity to resins.
3. Pregnant women or persons under the age of 18 are not recommended to undergo the procedure.
4. Terminate the whitening procedure if any patient reports severe tooth sensitivity. Remove all whitening gel before treating it with anti-sensitivity products. The whitening procedure should only be applied to intact and healthy enamel.

PRECAUTIONS

1. For professional in-office use only.
2. Operators must wear gloves.

SPECIAL NOTES

1. Advise the patient that it is safe to swallow their saliva during treatment.
2. If whitening gel comes into contact with the patient's gums during the treatment, the pink tissue may turn white during the treatment. This will fade in a few hours following the treatment and is not cause for concern. If conditions persist, contact your dentist.

STORAGE CONDITION

1. Store kit in a cool, dry place. Avoid exposure to direct sunlight.
2. BEYOND® Simplex™ Chemically Activated Formula Whitening Gel syringes MUST be stored in a refrigerator at 2-8°C.

IMPORTANT POINTS TO NOTE

1. The hydrogen peroxide whitening gel syringe has been laboratory tested to remain stable under 30 days of continual exposure to high temperatures (30-35°C). However, shipping conditions and storage vary or even short-term exposure to higher temperatures can damage material. Do not use if the following conditions are present:
 - Foil package around whitening gel syringes is no longer vacuum-sealed.
 - There is gel leakage evident
 - Gel is no longer viscous and is in a liquid state.
3. BEYOND® Simplex™ Chemically Activated Formula Whitening Gel contains hydrogen peroxide. Keep out of reach of children. If accidentally swallowed, get medical help or contact a Poison Control Center right away. If this product comes into contact with skin or eyes, rinse immediately with water and seek medical attention.

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