Table of Contents

- 1 English
- 2 Français
- 3 Español
- 4 Português
- 5 繁体中文

| Symbol | Meaning                                         | Symbol | Meaning      | Symbol | Meaning             |
|--------|-------------------------------------------------|--------|--------------|--------|---------------------|
|        | Consult Instructions for Use                    |        | Use By       |        | Date of Manufacture |
|        | Caution                                         |        | Batch Code   |        | Temperature Limit   |
|        | Authorized Representative in the European Union |        | Manufacturer |        | Do Not Reuse        |

1 EN

TREATMENT PROCEDURE

PRE-TREATMENT

Perform a pre-treatment oral exam and consult with the patient on his or her desired whitening results to determine the best treatment plan for his or her individual situation.

WHITENING TREATMENT

1. Apply lip protection cream and insert the BEYOND® FlexiBlue® Cheek Retractor with the tongue guard pointing down. Stretch the face protection cloth around the outside of the cheek retractor and fold down the top. Determine the existing tooth shade. (Fig. 1 & 2)
2. Place the protective eye goggles on the patient.
3. Polish teeth with the polishing sand provided. (Add water as necessary).
4. Insert cotton rolls on either side of the buccal frenum. Apply extra lip protection along the insides of the superior and inferior lip. (Fig. 3)
5. Dry the teeth and gums. Apply the BlueSeal gingival protection. Light cure in a circular motion. (Fig. 4 & 5)
6. Attach the mixing tip to the dual barrel syringe of whitening gel and apply a 2-3 mm layer of BEYOND II Advanced Formula Whitening Gel to the dry surface of the teeth. (Fig. 6)
7. Set the BEYOND® Whitening Accelerator for an eight minute cycle using the Set Time button on the control panel. Position the BEYOND® Whitening Accelerator lamp head at a 90° angle to the patient's teeth and press the Start/Stop button to begin the cycle (Fig 7).
8. At the end of the eight minute cycle, use suction to remove the whitening gel. DO NOT rinse with water. Repeat Steps 8-10 two more times for a total of three, eight minute treatment cycles.
9. After last cycle, suction off whitening gel, remove gingival protection and wipe teeth clean with cotton balls or gauze. Fluoride gel may be used at this point for one minute, if sensitivity occurs.

POST-TREATMENT

At the end of the treatment, suction off the fluoride and remove the face protection cloth, cheek retractor, cotton rolls, and protective eye goggles and ask the patient to rinse his or her mouth with warm water. Show the patient his or her new white tooth shade and give them a Post-Whitening Care Instruction Card and trial-sized tube of Pearl White Whitening Toothpaste with Advanced Formula. (Fig. 8)

TREATMENT PROCEDURE FOR HOME TRAY SYSTEM

1. Floss and brush your teeth gently with a toothpaste (such as Pearl White Toothpaste) making sure not to irritate the gums. Rinse your mouth with warm water.
2. Place a drop of BEYOND® CoreWhite® whitening gel into the tooth compartments for your teeth that can be seen when you smile in both the top and bottom trays (usually 8-10 teeth per tray). Each syringe has three treatment doses. Use up to one dose to fill both trays.
3. Fit the trays over your top and bottom teeth snugly.
4. Remove excess gel with a cotton bud or a dry toothbrush. When inserting the trays, take care to avoid pushing the gel up over the edge of the tray.
5. Wear the trays for one hour. Do not wear the whitening trays overnight.

IMPORTANT POINTS TO NOTE

- Pregnant or lactating women or persons under the age of 16 are not recommended to undergo the procedure.
- Terminate the whitening procedure if any patient reports severe tooth sensitivity and remove all whitening gel before treating with fluoride.

Note: Within the 24 hours after the procedure, patients should avoid colored food and drinks. Use of tobacco products should be limited.

CONTRAINDICATIONS

- BEYOND® II Advanced Formula Whitening Gel is contraindicated for patients with known allergy or chemical sensitivity to peroxides or glycols.
- Do not use BlueSeal Gingival Protection on patients with known sensitivity to resins. If dermatitis, rash, or any other allergic reaction develops, flush area with water and refer patient to a physician.

FOR BEST RESULTS

The recommended initial treatment schedule is three, eight minute applications of the whitening gel. If additional whitening results are desired for chair side treatment, an additional eight minute session can be performed 24 hours later, or the treatment can be split over two days (2 x 8 minutes on day one, 2 x 8 minutes on day two). There is enough whitening gel in the BEYOND® II Advanced Formula Whitening Gel syringe for four, eight minute applications.

NOTE

Caution patients not to over-use whitening trays as repeated follow-up whitening at home is a common and avoidable cause of sensitivity. Most patients require little or no immediate use of the tray material following the chair-side procedure, but can instead use the custom trays and included take-home gel for once-monthly maintenance.

SPECIAL NOTES

- Advise the patient that it is safe to swallow their saliva during the treatment.
- If the whitening gel comes into contact with the patient's gums during the treatment, the pink tissue may turn white during the treatment. This will fade in a few hours following the treatment and is not cause for concern.
- Depending on the patient's diet, oral hygiene, and use of tobacco products, periodic touch-up treatments may be necessary to maintain his or her new, white smile.

STORAGE AND MAINTENANCE

- Store all items in a cool, dry place. Avoid exposure to direct sunlight.
- BEYOND® II Advanced Formula Whitening Gel syringes must be stored near 5°C. These syringes are packed separately for easy storage in the refrigerator.

PRECAUTIONS AND WARNINGS

The 35% hydrogen peroxide gel syringe has been laboratory tested to remain stable under 30 days continual exposure to high temperatures (30-35°C). However shipping conditions vary and even short-term exposure to higher temperatures can damage the material. Do not use if the following conditions are present:

- Foil package around whitening gel syringes is no longer vacuum-sealed
- There is gel leakage evident
- The gel is no longer viscous and is in a liquid state
- Syringe plunger is being pushed out of the syringe body

The BEYOND® II Advanced Formula Whitening Gel contains 35% hydrogen peroxide. Keep out of reach of children. If accidentally swallowed, get medical help or contact a Poison Control Center right away. If this product comes into contact with skin or eyes, rinse immediately with water and seek medical attention.

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