Diagram 1
"Actual Size"

Table of Contents

- | | | |
|-------------|--------------------|---------------|
| 1. English | 7. Français | 13. Português |
| 2. Bosanki | 8. Latviešu valoda | 14. Русские |
| 3. Čeština | 9. Lietuvių kalba | 15. Română |
| 4. Deutsch | 10. Magyar | 16. suomi |
| 5. ελληνικά | 11. Nederlands | 17. 中文 |
| 6. Español | 12. Polski | |

beyond[®] CoreWhite[®] 
Take-Home Whitening Kit

Check your package to ensure that the following items are all included:

- 6 x Syringes of BEYOND® CoreWhite® Whitening Gel (2.2mL each)
- 2 x Custom-made Impression Trays (provided by your dentist)
- 1 x BEYOND® CoreWhite® Protective Case for Whitening Trays
- 1 x BEYOND® CoreWhite® Protective Case for 6 Syringes of the Whitening Gel
- 1 x Instruction Manual

TREATMENT PROCEDURE

1. Floss and brush your teeth gently with a toothpaste (such as BEYOND® Pearl White® Toothpaste) making sure not to irritate the gums. Rinse your mouth with warm water.
2. Place a drop of BEYOND® CoreWhite® whitening gel (see Diagram 1 for size reference) into the tooth compartments for your teeth that can be seen when you smile in both the top and bottom trays (usually 8-10 teeth per tray). Each syringe has three treatment doses. Use up to one dose to fill both trays. (Fig. 1)
3. Fit the trays over your top and bottom teeth snugly. (Fig. 2 & 3)
4. Remove excess gel with a cotton bud or a dry toothbrush. When inserting the trays, take care to avoid pushing the gel up over the edge of the tray.
5. Wear the trays for 30 minutes. Do not wear the whitening trays overnight.
6. At the end of the 30 minutes, remove the trays. Rinse your mouth and brush excess gel off of your teeth. Rinse the trays with cold water. If necessary, use a toothbrush to remove any leftover gel. Place trays in the protective case and store in a cool, dry place. (Fig. 4)

CONTRAINDICATIONS

- BEYOND® CoreWhite® Whitening Gel is contraindicated for patients with known allergy or chemical sensitivity to peroxides or glycols.

FOR BEST RESULTS

The recommended initial treatment schedule is to use BEYOND® CoreWhite® on 14 consecutive days for 30 minutes per day. Please consult with your dentist to determine if an alternative or additional treatment schedule is necessary for your individual situation.

Minimize consumption of colored food and drinks while using BEYOND® CoreWhite®. Try to limit use of tobacco products. Maintain good oral hygiene during the treatment.

SPECIAL NOTES

- If the whitening gel comes into contact with the gums during the treatment, the pink tissue may turn white during the treatment. This will fade in a few hours following the treatment and is not cause for concern.
- If you have severe stains caused by tetracycline, fluoride, or another factor, discuss your best options for whitening these stains with your dentist. BEYOND® CoreWhite® is effective on whitening these types of stains; however additional treatment cycles and adaptations may be necessary to achieve best results.

It is recommended that you consult with your dental professional before beginning any new treatment.

Do not use if syringes are leaking.

Keep out of reach of children. Not suitable for pregnant or lactating women or persons under 18 years of age.

POST-TREATMENT

It is recommended that you use BEYOND® Pearl White® Toothpaste to maintain your new, white smile. For a healthy, shiny, white smile: Brush twice a day, floss daily, and visit your dentist regularly. Depending on your diet, oral hygiene, and use of tobacco products, periodic touch-up treatments may be necessary to maintain your new , white smile.

STORAGE AND MAINTENANCE

- Store all items in a cool, dry place. Avoid exposure to heat or direct sunlight.

BEYOND® CoreWhite® Whitening Gel

Active ingredients: Hydrogen peroxide (6%)

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MDSS GmbH