

EN



P/N	DESCRIPTION	SIZE
BY-PD01205M	BEYOND® FlexiBlue® Cheek Retractor M - 5 Pack	M
BY-PD01205L	BEYOND® FlexiBlue® Cheek Retractor L - 5 Pack	L

DESCRIPTION & INDICATIONS:

The BEYOND® FlexiBlue® Comfort Cheek Retractor is indicated as an in-office dental device intended for increasing the work area of the mouth in which oral surgery can be performed.

PRODUCT FEATURES:

The BEYOND® FlexiBlue® Comfort Cheek Retractor contains a pair of channeled edges that can be inserted around the lips of the patient, and a projected bridge connecting the edges at each side, by which the work area is defined. The user can either pull or push the retractor in order to expose the desired side of the mouth, making it a one-handed device.

BEYOND® FlexiBlue® Comfort Cheek Retractor is designed to enlarge the vestibular fundus and increase the work area of the anterior 16-20 teeth. The material and elasticity put patients in a comfortable, safe condition for treatment. The device can be cold solution sterilized.

TREATMENT PROCEDURE:

1. Apply lip protection cream prior to inserting cheek retractor.
2. Insert cheek retractor with tongue guard pointing downwards as illustrated.

COMPOSITION: Polypropylene

PRECAUTIONS: Discard broken, defective or worn-out products.

CLEANING NOTE:

Immediately after use, thoroughly rinse fluids / debris from the device with copious amounts of cool running water. Clean with soap and water. Visually check to ensure surfaces are clean and free from damage, stains and bioburden. Rinse retractors with distilled or de-mineralized water if possible. Inspect device for visual damage, wear and tear or deterioration.

STERILIZATION: Use cold solution for sterilization.

STORAGE AND HANDLING:

Keep the product in a designated and closed cabinet protected from dust and insects.

PACKAGE CONTENTS:

BEYOND® FlexiBlue® Comfort Cheek Retractor Kit
5 x FlexiBlue® Comfort Cheek Retractors