Table of Contents

- 1 English
- 2 Français
- 3 Español
- 4 Português
- 5 繁体中文

BEYOND® FlexiBlue® Cheek Retractors are available in packs of five from BEYOND®. They are not included in the Max5 Treatment Kits. Please contact your BEYOND® products dealer to order.

Symbol	Meaning	Symbol	Meaning	Symbol	Meaning
	Consult Instructions for Use		Use By		Date of Manufacture
	Caution		Batch Code		Temperature limit
	Keep away from sunlight		Manufacturer		

1 EN

Treatment Procedure (Please see the instruction manual included with the BEYOND® Whitening Accelerator for more detailed instructions.)

PRE-TREATMENT

Step 1: Perform a pre-treatment oral exam and consult with the patient on his or her desired whitening results to determine the best treatment plan for his or her individual situation.

WHITENING TREATMENT

- Step 2: Apply lip protection cream and insert the BEYOND® FlexiBlue® Cheek Retractor. Determine the existing tooth shade. (see Fig. 1 & 2)
- Step 3: Place the protective eye goggles on the patient.
- Step 4: Insert cotton rolls on either side of the frenum of the upper and lower lips. Apply extra lip protection cream if necessary. (see Fig. 3)
- Step 5: Dry the teeth and gums. Apply the BEYOND® BlueSeal® Gingival Protection. Light cure in a circular motion. (see Fig. 4 & 5)
- Step 6: Attach the mixing tip to the dual barrel syringe of whitening gel and apply a 2-3 mm layer of BEYOND® Whitening Gel to the dry surface of the teeth. (see Fig. 6)
- Step 7: Position the BEYOND® Whitening Accelerator lamp head at a 90° angle to the patient's teeth and press the Start button to begin the first 10 minute cycle. (see Fig. 7)
- Step 8: At the end of the 10 minute cycle, use dry suction to remove the whitening gel. DO NOT rinse with water. Repeat Steps 5-7, two more times for a total of three, 10 minute treatment cycles.

POST-TREATMENT

- Step 9: At the end of the last 10 minute cycle, remove the gel, gingival protection, cheek retractor, gauze, and protective eye goggles and ask the patient to rinse his or her mouth with warm water.
- Step 10: Show the patient his or her new white tooth shade. (see Fig. 8)

IMPORTANT POINTS TO NOTE

Pregnant women or persons under the age of 18 are not recommended to undergo the procedure. Terminate the whitening procedure if any patient reports severe tooth sensitivity and remove all whitening gel before treating with fluoride.

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