
Five-Patient Professional Teeth Whitening Kit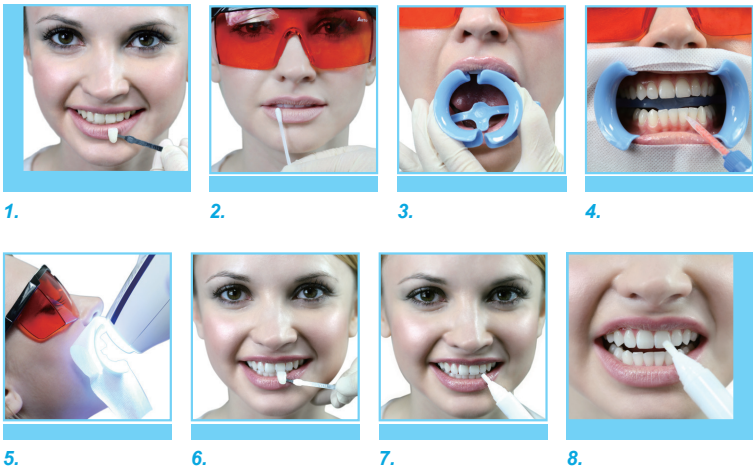

Multilingual Treatment Guide

Table of Contents

- 1 English
- 2 Français
- 3 Español
- 4 Português
- 5 繁体中文

Symbol	Meaning	Symbol	Meaning	Symbol	Meaning
	Consult Instructions for Use		Use By		Date of Manufacture
	Caution		Batch Code		Temperature Limit
	Authorized Representative in the European Union		Manufacturer		Do Not Reuse

1EN

PRE-TREATMENT

Step 1: Perform a pre-treatment oral exam and consult with the patient on his or her desired whitening results to determine the best treatment plan for his or her individual situation.

TREATMENT

Step 2: Determine the existing tooth shade. (see Fig. 1)
Step 3: Apply lip protection cream and place the protective eye goggles on the patient. (see Fig. 2)
Step 4: Insert the BEYOND® FlexiBlue® Cheek Retractor. (see Fig. 3)
Step 5: Dry the teeth and gums.
Step 6: Attach the mixing tip to the dual barrel syringe of whitening gel and apply a 2-3 mm layer of Osmo™ Whitening Gel to the dry surface of the teeth. (see Fig. 4)
Step 7: Position the BEYOND® Accelerator lamp head at a 90° angle to the patient's teeth and press the Start button to begin the first 12 minute cycle. (see Fig. 5)
Step 8: At the end of the 12 minute cycle, use dry suction to remove the whitening gel. DO NOT rinse with water. Repeat Steps 5-7, two more times for a total of three, 12 minute treatment cycles.

POST-TREATMENT

Step 9: At the end of the last 12 minute cycle, remove the gel, cheek retractor, gauze, and protective eye goggles and ask the patient to rinse his or her mouth with water.
Step 10: Show the patient his or her new white tooth shade. (see Fig. 6)

Osmo™ Whitening Pen Instructions

Step 1: Remove cap from Osmo™ Whitening Pen.
Step 2: Press the button at the bottom of the pen (by clicking), until a droplet is seen in the brush of the pen (first time use may require about 15-20 clicks before droplet appears).
Step 3: If not using cheek retractor, have patient smile wide, keeping lips away from the teeth. Apply a thin layer of Osmo™ Whitening Pen gel directly onto the front surface of the teeth, avoiding gums or lips. Keep mouth open for 30 seconds. (see Fig. 7 & 8)
Step 4: Mouth can be rinsed after ten minutes.
Step 5: Replace the cap when finished.

IMPORTANT POINTS TO NOTE

Pregnant women or persons under the age of 18 are not recommended to undergo the procedure.
Terminate the whitening procedure if any patient reports severe tooth sensitivity and remove all whitening gel before treating with fluoride.
For home use of Whitening Pen, for best whitening results use at night as saliva output is at lowest level.
Do not eat or drink within 30 minutes after using whitening pen.

Contraindications

Osmo™ Whitening Gel is contraindicated for patients with known allergy or chemical sensitivity to peroxides or glycols.

For better comfort, applying lip balm to lips before whitening pen treatment is suggested.

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