Five-Patient Professional Teeth Whitening Kit


1.

2.

3.

4.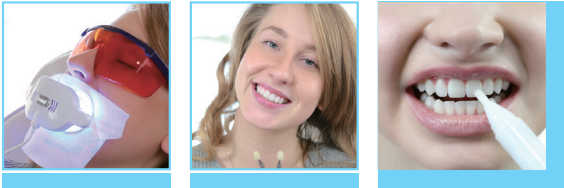

5.

6.

7.

Symbol	Meaning	Symbol	Meaning	Symbol	Meaning
	Consult Instructions for Use		Use By		Date of Manufacture
	Caution		Batch Code		Temperature limit
	Authorized Representative in the European Union		Manufacturer		Do Not Reuse

Table of Contents

- 1 English
- 2 Français
- 3 Español
- 4 Português
- 5 繁体中文

1EN

PRE-TREATMENT

Step 1: Perform a pre-treatment oral exam and consult with patient on desired whitening results to determine the best treatment plan for individual situation.

TREATMENT

Step 2: Determine existing tooth shade. (see Fig. 1)
Step 3: Apply lip protection cream and place protective eye goggles on patient. (see Fig. 2-3)
Step 4: Insert BEYOND® Cheek Retractor and apply face cloth.
Step 5: Dry the teeth and gums.
Step 6: Attach mixing tip to dual barrel syringe of whitening gel and apply a 2-3 mm layer of Osmo® Plus whitening gel to dry surface of teeth. Avoid contact with gingiva when applying the whitening gel. If necessary, apply lip cream to avoid contact with gums. (see Fig. 4)
Step 7: If using Polus® Whitening Accelerator, position lamp head at a 90° angle to teeth and press Start button to begin first cycle. See instruction manual for details. If using BEYOND® II or BEYOND® II Ultra Whitening Accelerator, attach the light head onto cheek retractor and press the button to begin first cycle. See instruction manual for details. (see Fig. 5)
Step 8: At the end of cycle, remove whitening gel via suction or cotton bud. DO NOT rinse with water. Repeat Steps 5-7 two more times for a total of three, 12 minute treatment cycles.

POST-TREATMENT

Step 9: At end of last 12 minute cycle, remove gel, cheek retractor, gauze and protective eye goggles and instruct patient to rinse mouth with warm water.
Step 10: Show patient new white tooth shade. (see Fig. 6)

OSMO® WHITENING PEN INSTRUCTIONS

Step 1: First, start by brushing your teeth with toothbrush. (BEYOND® soft-bristle toothbrush with bamboo handle is recommended.)
Step 2: Dry teeth surface with tissue or cotton ball.
Step 3: Remove the top cap of the pen and twist the bottom of the pen until you see the gel on the brush tip.
Step 4: Then, apply a thin layer of whitening gel to each tooth's surface (upper and lower 6-8 teeth each), making sure the tooth's surface is in contact with whitening gel. Avoid getting any of the whitening product on your gums and lips.
Step 5: After application, leave your mouth open with your lips away from the treated teeth for 30 seconds to 1 minute. Try to avoid swallowing any gel.
Step 6: Rinse out whitening gel with warm water after 10 minutes. For best results, recommended treatment schedule is to use BEYOND® Osmo® Pen for 7-10 consecutive days. (see Fig. 7)

IMPORTANT POINTS TO NOTE

Pregnant women or persons under the age of 18 are not recommended to undergo procedure.
Terminate whitening procedure if any patient reports severe tooth sensitivity and remove all whitening gel before treating with fluoride. Do not use overnight.
Do not sleep while using whitening pen.
Do not eat or drink within 30 minutes after using whitening pen.
Do not swallow whitening gel while using whitening pen.

CONTRAINDICATIONS

The Osmo® Whitening Gel is contraindicated for patients with known allergy or chemical sensitivity to peroxides or glycols.
For better comfort, applying lip balm to lips before whitening pen treatment is suggested.

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