Table of Contents

1 English

2 Français

3 Español

4 Português

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EN 1

Treatment Procedure (Please see the instruction manual included with BEYOND® Whitening Accelerators for more detailed instructions.)

PRE-TREATMENT

Step 1: Perform a pre-treatment oral exam and consult with patient on his or her desired whitening results to determine best treatment plan for his or her individual situation.

WHITENING TREATMENT

Step 2: Apply lip protection cream and insert cheek retractor (BEYOND® FlexiBlue® Cheek Retractor is recommended). Attach face protection cloth. Determine existing tooth shade. (see Fig. 1 & 2)

Step 3: Place protective eye goggles on patient.

Step 4: Insert cotton rolls on either side of the frenum of upper and lower lips. Apply extra lip protection cream if necessary. (see Fig. 3)

Step 5: Dry teeth and gums. Apply BEYOND® BlueSeal® Gingival Protection. Light cure in a circular motion for around 30 seconds. (see Fig. 4 & 5)

Step 6: Attach mixing tip to dual barrel syringe and apply a 2-3 mm layer of BEYOND® Advanced Formula Whitening Gel to dry surface of front upper and lower sixteen teeth. (see Fig. 6)

Step 7: Position BEYOND® Whitening Accelerator lamp head at a 90° angle as close to patient's teeth as possible. Press Start button to begin first 10 minute cycle. (see Fig. 7) *Note: Treatment time can be adjusted according to patient's teeth condition and dentist's advice.

Step 8: At the end of 10 minute cycle, suction off whitening gel. DO NOT rinse with water. Repeat Steps 6-7, two more times for a total of three, 10 minute treatment cycles.

POST-TREATMENT

Step 9: At the end of last 10 minute cycle, remove gel, gingival protection, cheek retractor, cotton rolls, and protective eye goggles. Instruct patient to rinse his or her mouth with warm water.

Step 10: Show patient his or her new white tooth shade. (see Fig. 8)

CONTRAINDICATIONS

1. BEYOND® Advanced Formula Whitening Gel is contraindicated for patients with known allergy or chemical sensitivity to peroxides or glycols.

2. Do not use BEYOND® BlueSeal® Gingival Protection on patients with known sensitivity to resins.

3. Pregnant women or persons under the age of 18 are not recommended to undergo the procedure.

4. Terminate whitening procedure if any patient reports severe tooth sensitivity and remove all whitening gel before treating with antisensitivity products. Whitening procedure should be applied in intact and healthy enamel conditions.

SPECIAL NOTES

1. Advise patient that it is safe to swallow their saliva during treatment.

2. If whitening gel comes into contact with patient's gums during the treatment, the pink tissue may turn white during the treatment.

This will fade in a few hours following the treatment and is not cause for concern. If conditions persist, contact your dentist.

STORAGE CONDITION

1. Store kit in a cool, dry place. Avoid exposure to direct sunlight.

2. BEYOND® Advanced Formula Whitening Gel syringes MUST be stored in a refrigerator at 2-8°C.

IMPORTANT POINTS TO NOTE

1. The 35% hydrogen peroxide gel syringe has been laboratory tested to remain stable under 30 days continual exposure to high temperatures (30-35°C).

2. However, shipping conditions and storage vary or even short-term exposure to higher temperatures can damage material. Do not use if the following conditions are present:

• Foil package around whitening gel syringes is no longer vacuum-sealed.

• There is gel leakage evident

• Gel is no longer viscous and is in a liquid state.

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